


## How to Setup Remote Exam Device Control (Remote Medic Nurse)

Jay-Ileen (Ai) - 2024-11-18 - VSee Messenger (Providers and Patients)

Before the first appointment, you need to have the one-time set-up of the Peripheral Streaming Devices for audio and video.

### One-time Set-up of Remote Audio

1. Click on the **gear** icon or **Settings**.
2. Select **Audio**.
3. Click on + **Add Microphone**.

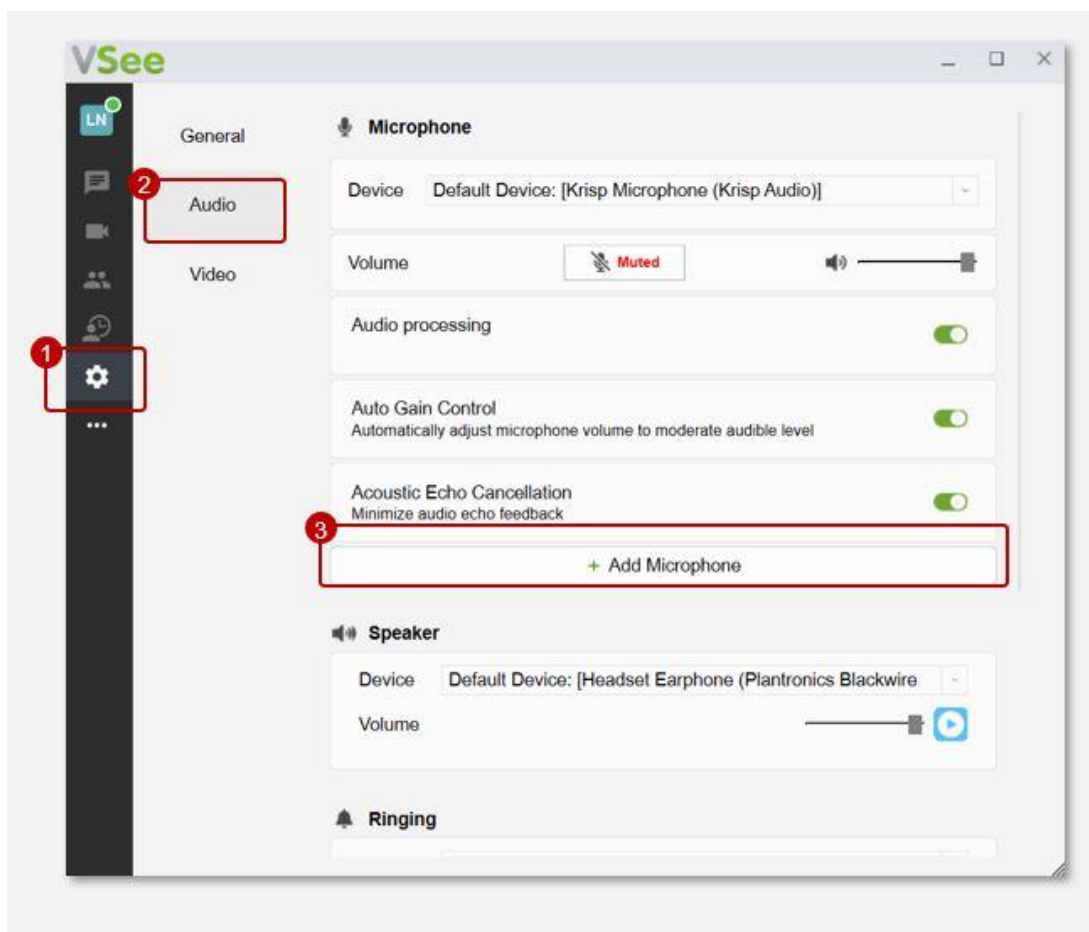




1. Select which type of audio device you want to add:

- **Aux Mic**

- **Stethoscope**

2. Under **Device**, select from the dropdown options the name of the device you connected to the computer/console.

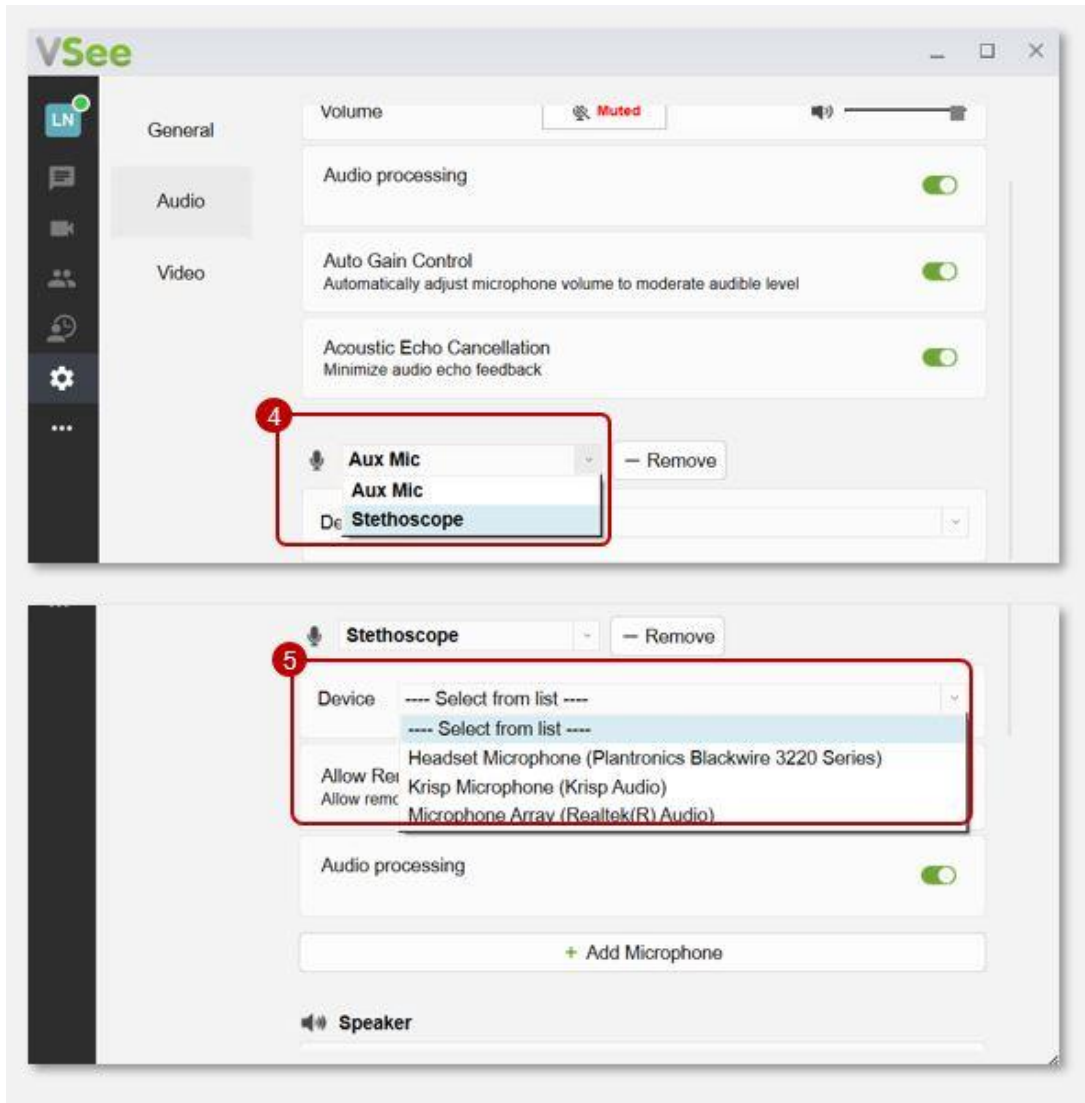




1. Under **Allow Remote**, select **Only Moderators** from the dropdown options.

(This option allows the **host** or the Provider to be the only one to turn on/off the device.

1. Under **Audio processing**, click to toggle on.

(This is recommended for clear audio for Stethoscope mode)

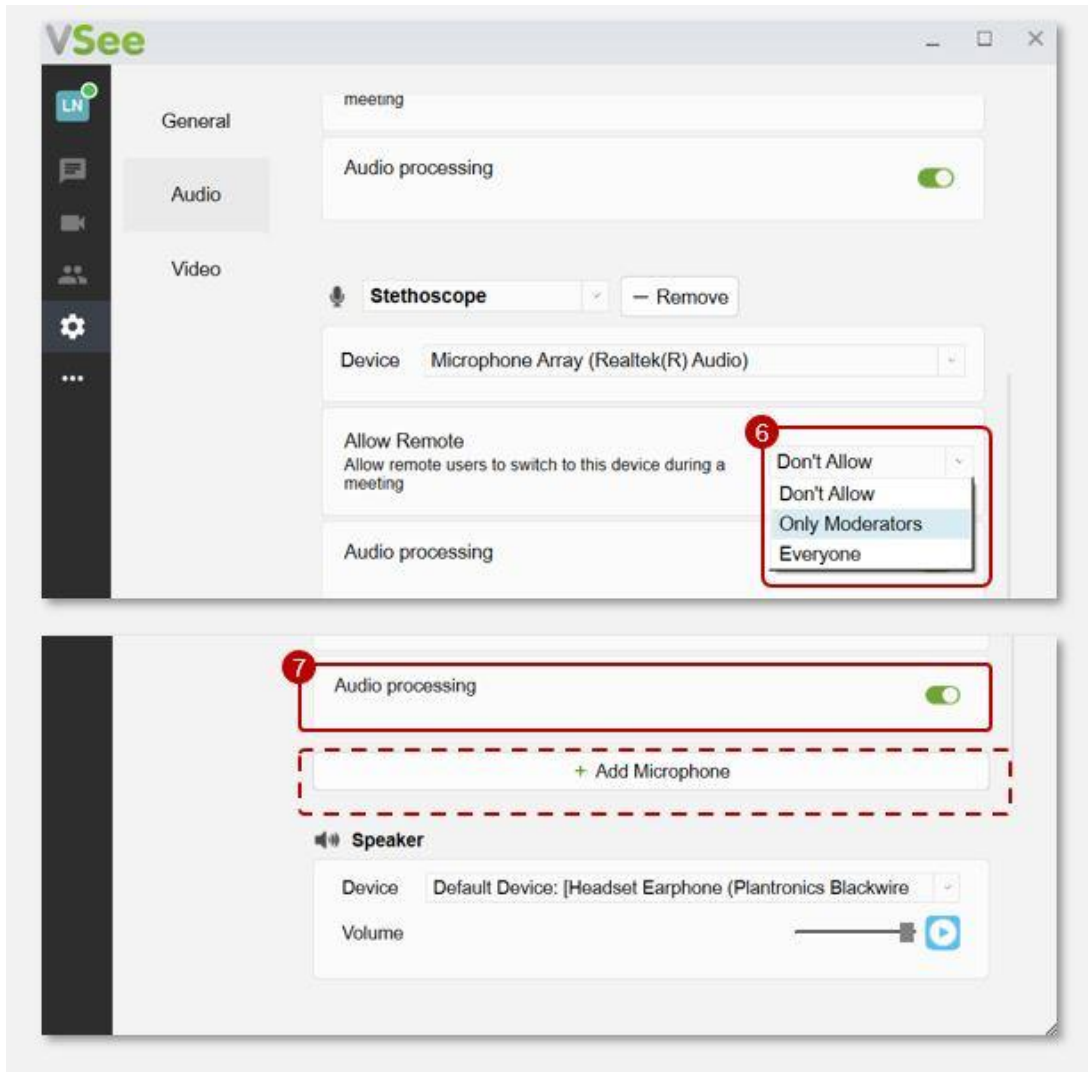




\*Scroll down and click on a new **+Add Microphone** to add more audio peripheral streaming devices. You need to add them one by one.

*Note: You need to turn on the main microphone of the patient during the visit so that the Provider can remotely access it.*

#### **One-time Set-up of Remote Camera**

1. Click on the **gear** icon or **Settings**.
2. Select **Video**.
3. Click on **+ Add Camera**.

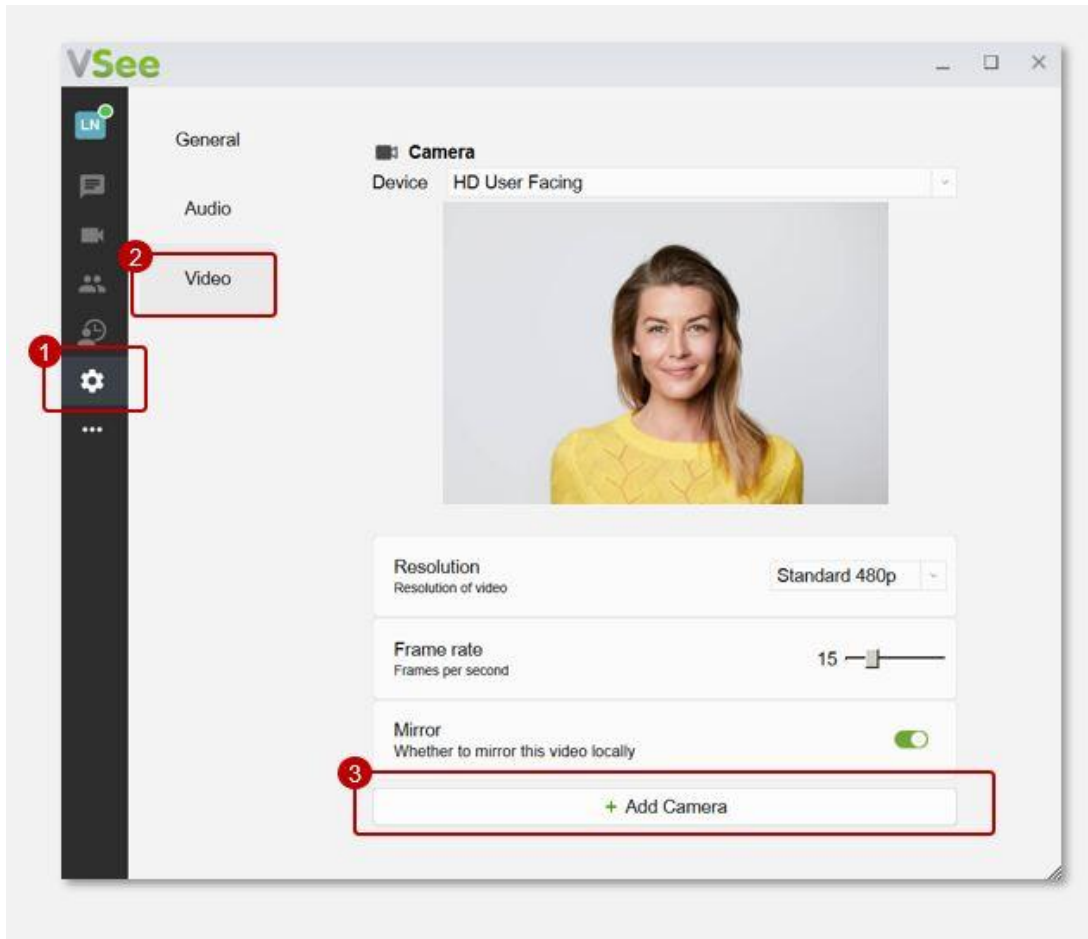




1. Select which type of camera device you want to add:

- Webcam
- General Exam Camera
- Otoscope
- Iriscope
- Dermatoscope
- Ultrasound

2. Under **Device**, select from the dropdown options the name of the device you connected to the computer/console.



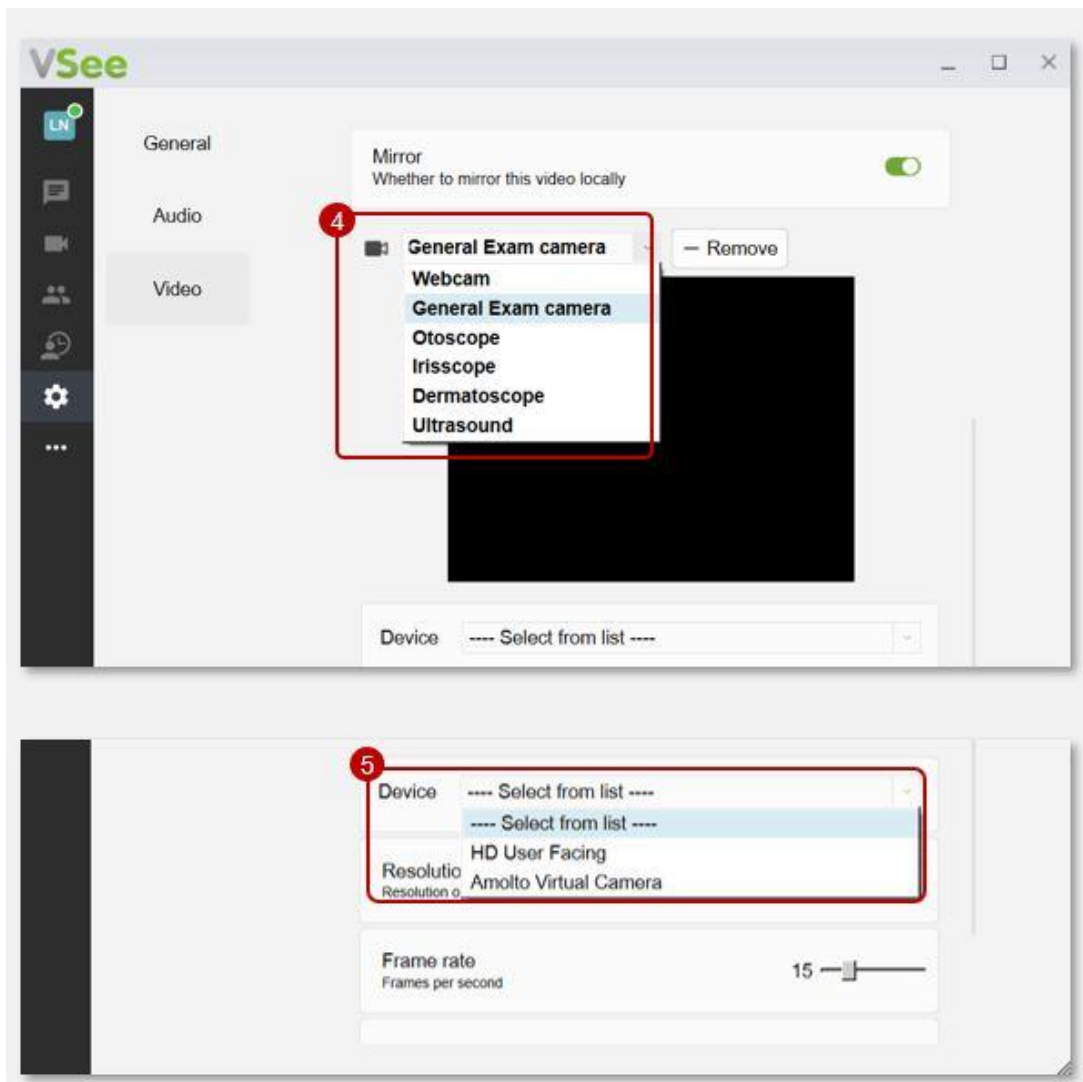




1. Under **Allow Remote**, select **Only Moderators** from the dropdown options.

(This option allows the **host** or the Provider to be the only one to turn on/off the device.

Under **Enable**, do not toggle on so that the video feed of the added camera will not automatically show once the visit starts. The Remote Provider will turn it on when needed.



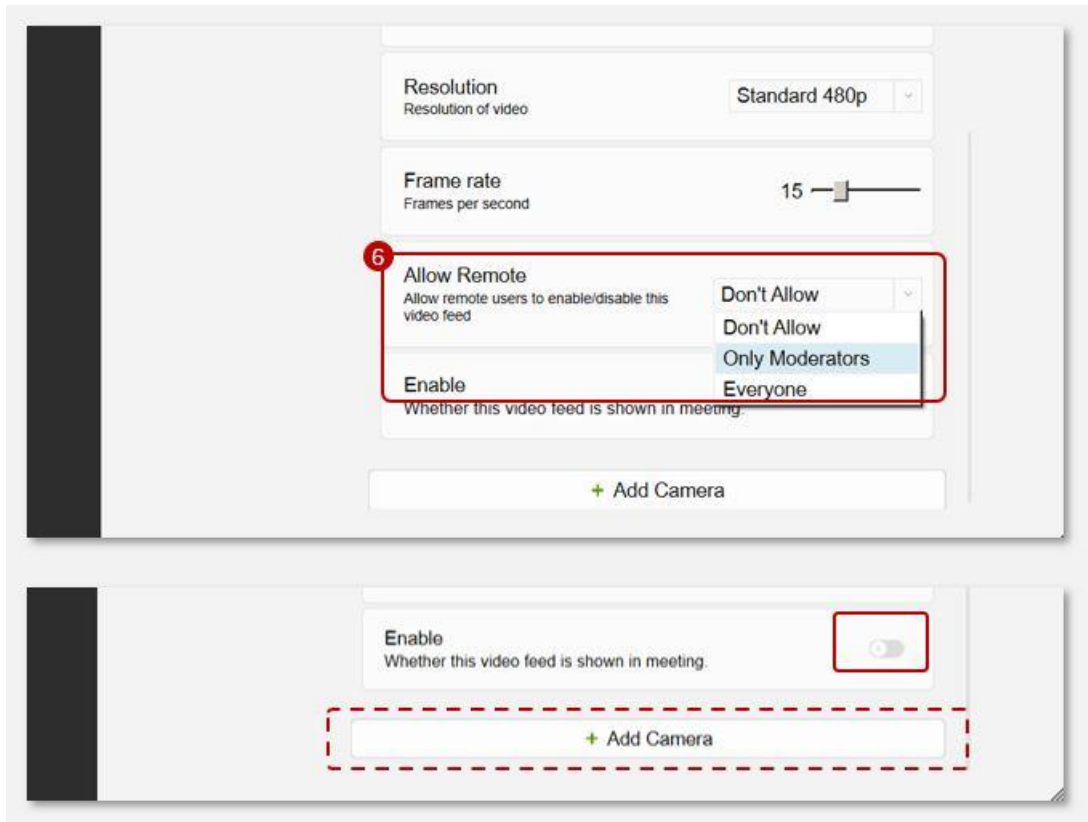

\*Scroll down and click on a new **+Add Camera** to add more camera peripheral streaming devices. You need to add them one by one.

*Note: You need to turn on the main camera of the patient during the visit so that the Provider can remotely access it.*

Related Articles:

- [Remote Exam Device Control](#)
- [How to Use the Remote Exam Device Control](#)

Need more assistance? Send us an email at [help@vsee.com](mailto:help@vsee.com).

Tags  
control device  
peripheral device  
peripheral streaming devices  
Remote Exam Device  
Remote Exam Device Control