


How to Setup Remote Exam Device Control (Remote Medic Nurse)

Jay-Ileen (Ai) - 2024-11-18 - [VSee Messenger \(Providers and Patients\)](#)

Before the first appointment, you need to have the one-time set-up of the Peripheral Streaming Devices for audio and video.

One-time Set-up of Remote Audio

1. Click on the **gear** icon or **Settings**.
2. Select **Audio**.
3. Click on **+ Add Microphone**.

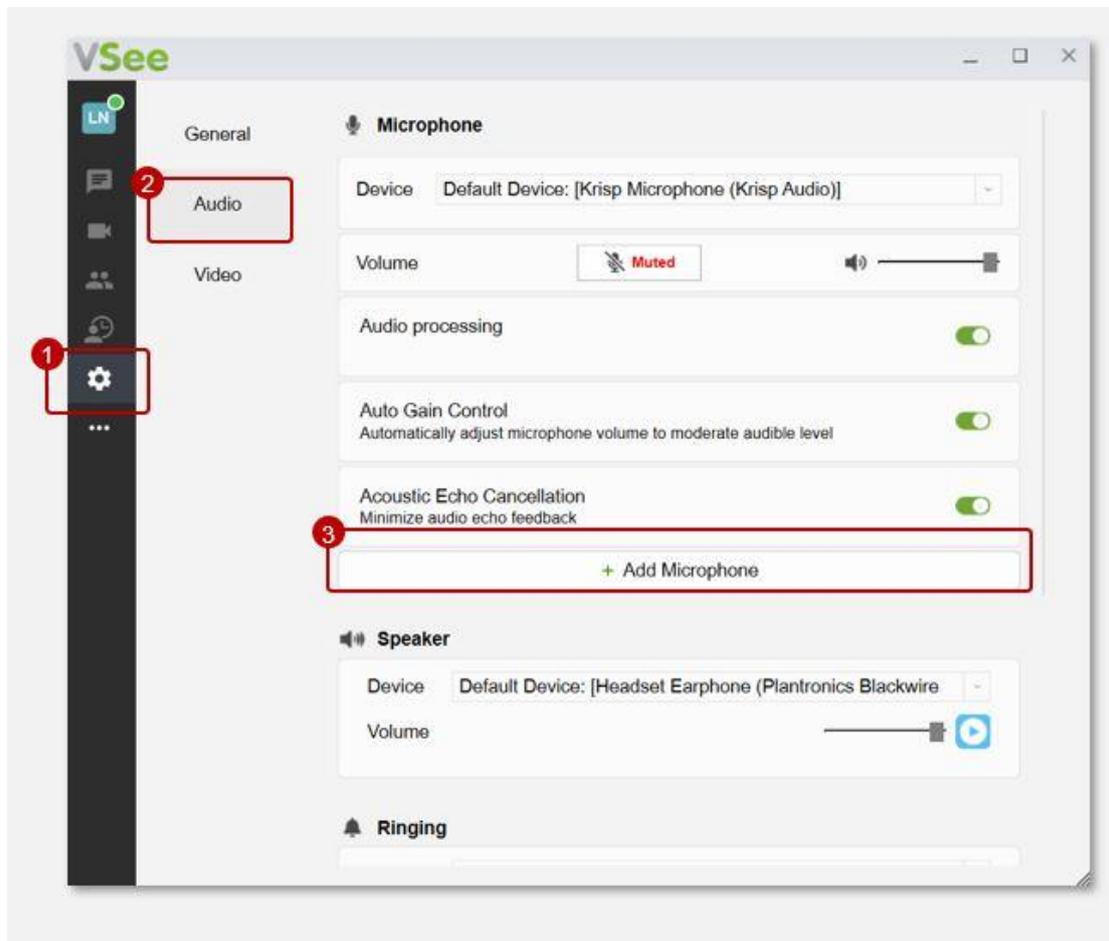




1. Select which type of audio device you want to add:

- **Aux Mic**

- **Stethoscope**

2. Under **Device**, select from the dropdown options the name of the device you connected to the computer/console.

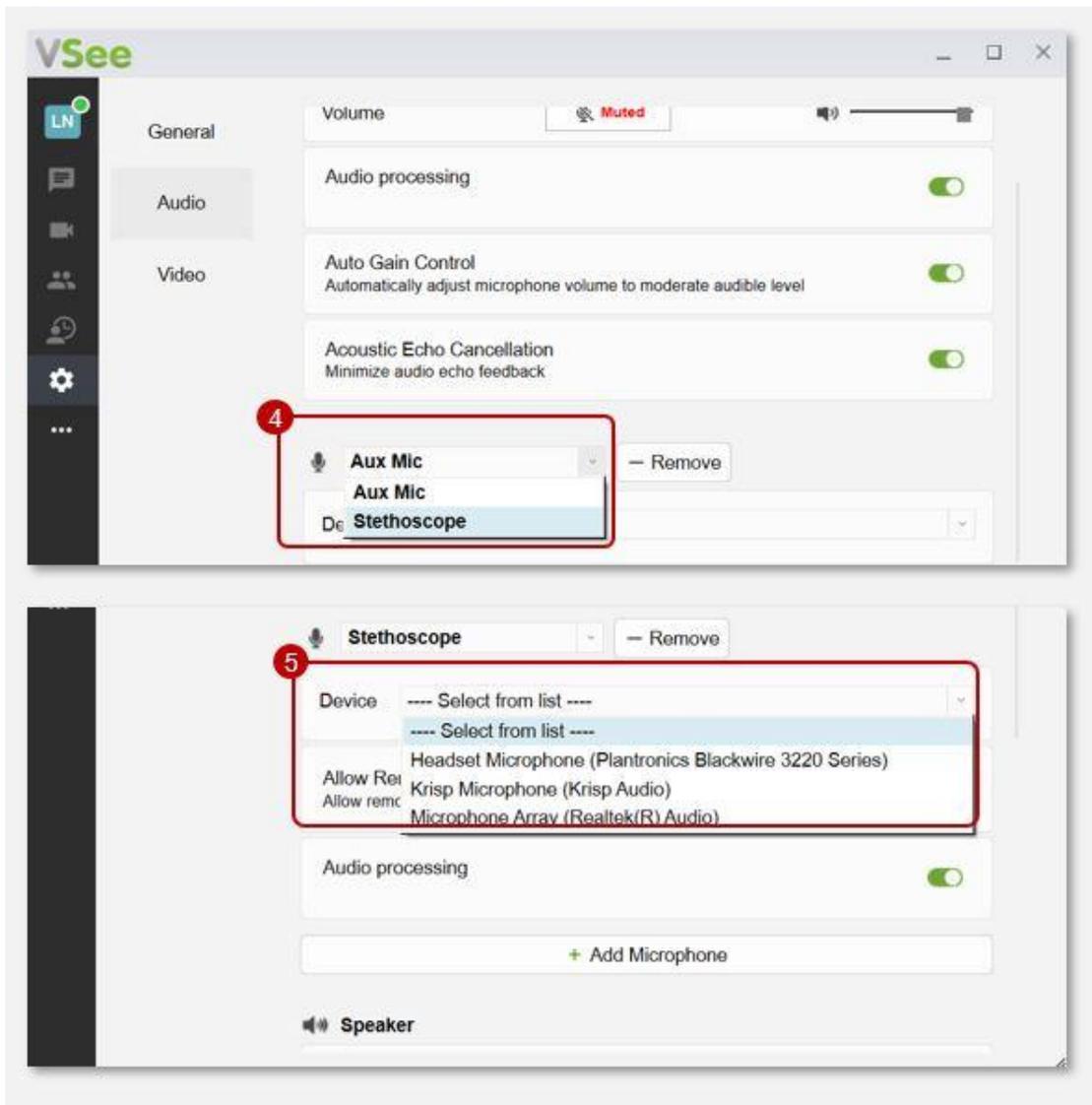




1. Under **Allow Remote**, select **Only Moderators** from the dropdown options.

(This option allows the **host** or the Provider to be the only one to turn on/off the device.)

1. Under **Audio processing**, click to toggle on.

(This is recommended for clear audio for Stethoscope mode)

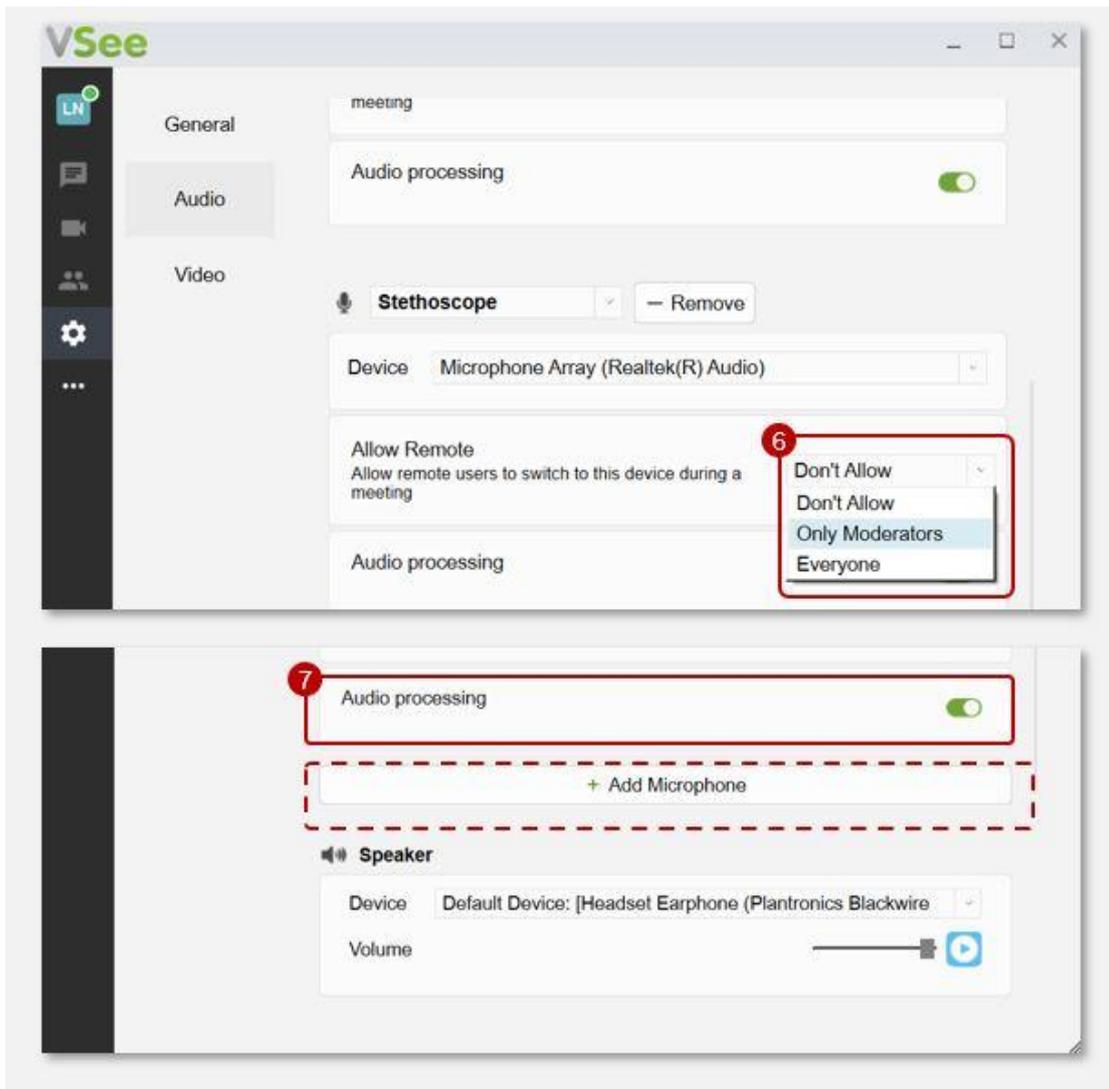




*Scroll down and click on a new **+Add Microphone** to add more audio peripheral streaming devices. You need to add them one by one.

Note: You need to turn on the main microphone of the patient during the visit so that the Provider can remotely access it.

One-time Set-up of Remote Camera

1. Click on the **gear** icon or **Settings**.
2. Select **Video**.
3. Click on **+ Add Camera**.

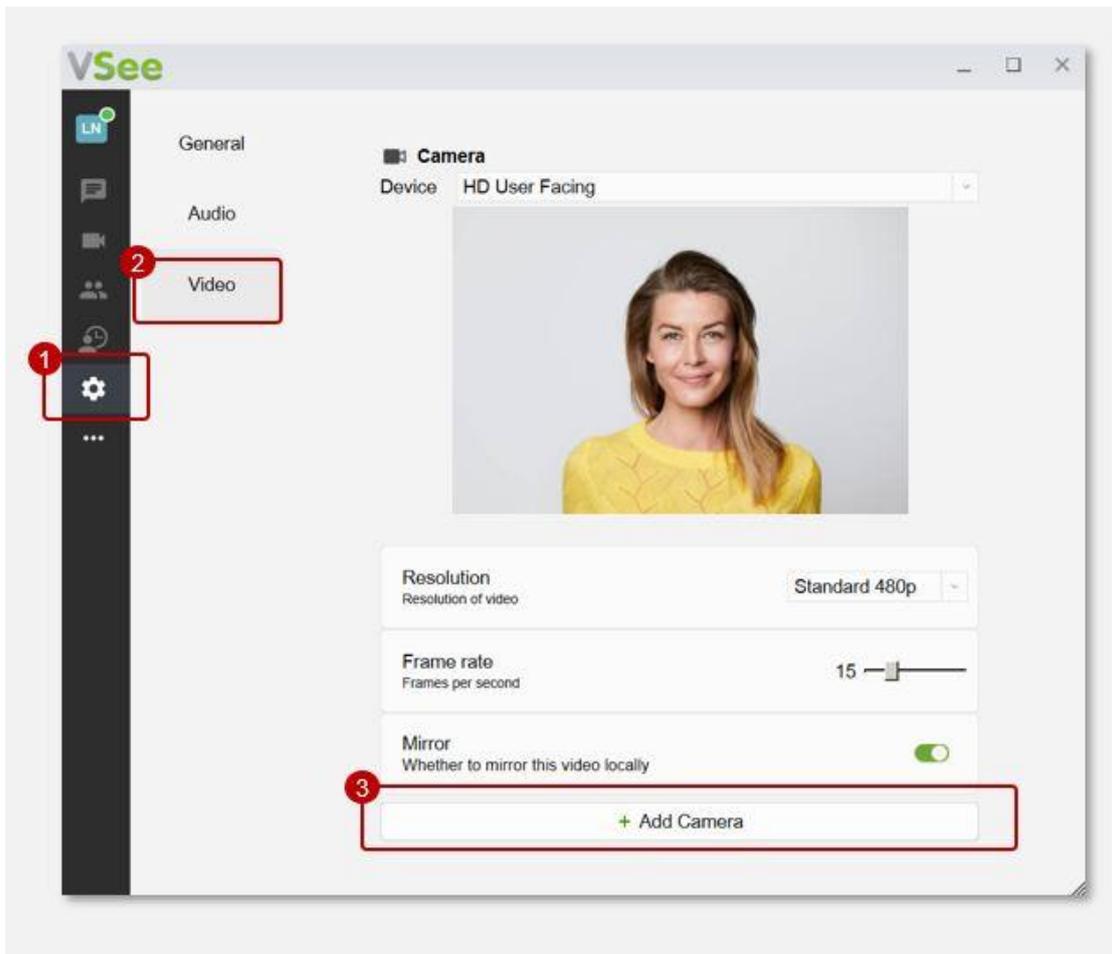




1. Select which type of camera device you want to add:

- Webcam
- General Exam Camera
- Otoscope
- Iriscope
- Dermatoscope
- Ultrasound

2. Under **Device**, select from the dropdown options the name of the device you connected to the computer/console.

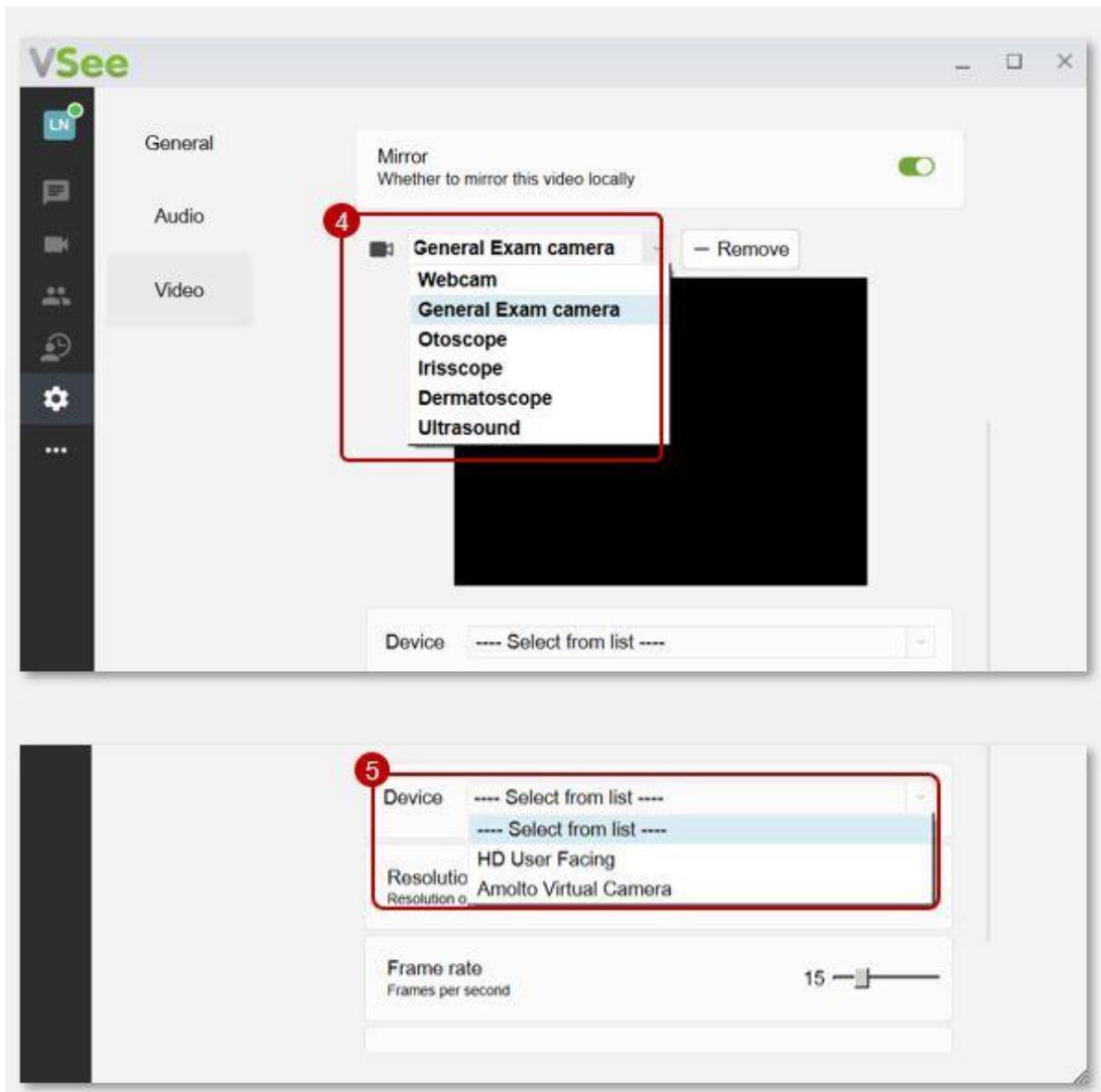




1. Under **Allow Remote**, select **Only Moderators** from the dropdown options.

(This option allows the **host** or the Provider to be the only one to turn on/off the device.

Under **Enable**, do not toggle on so that the video feed of the added camera will not automatically show once the visit starts. The Remote Provider will turn it on when needed.



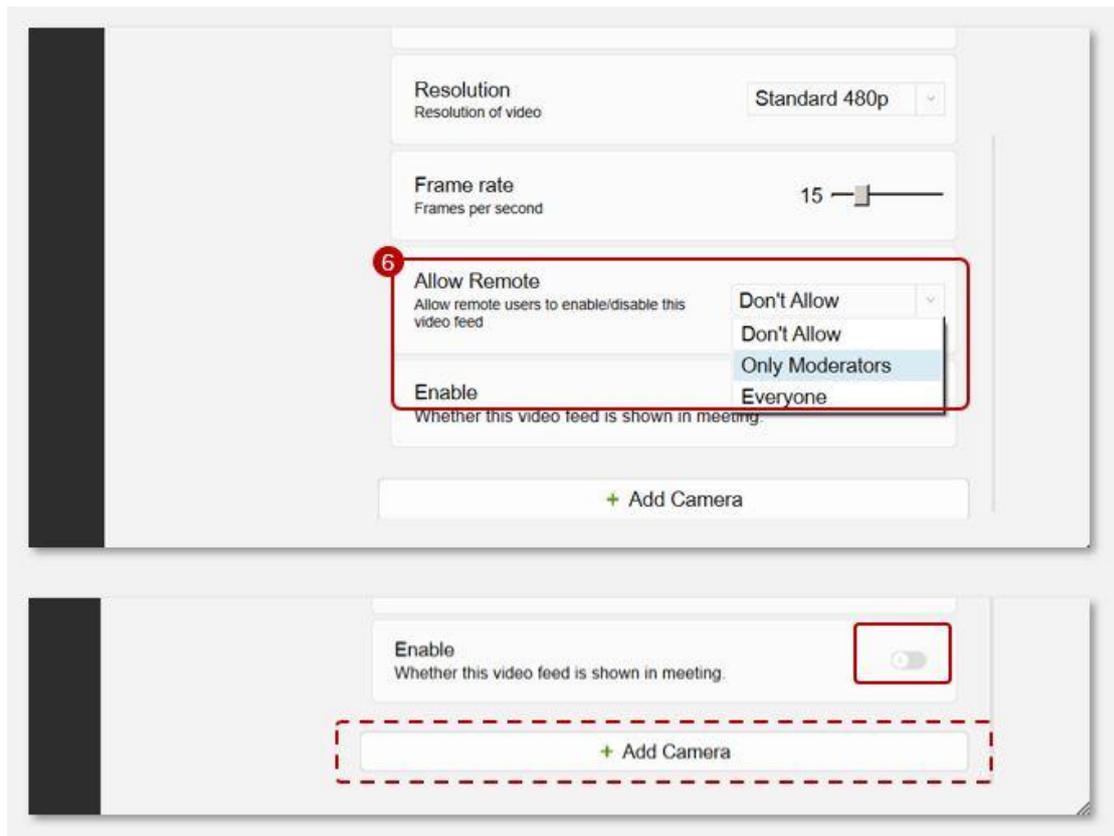

*Scroll down and click on a new **+Add Camera** to add more camera peripheral streaming devices. You need to add them one by one.

Note: You need to turn on the main camera of the patient during the visit so that the Provider can remotely access it.

Related Articles:

- [Remote Exam Device Control](#)
- [How to Use the Remote Exam Device Control](#)

Need more assistance? Send us an email at help@vsee.com.

- Tags
- [control device](#)
- [peripheral device](#)
- [peripheral streaming devices](#)
- [Remote Exam Device](#)
- [Remote Exam Device Control](#)