

Stethoscope Mode in VSee Messenger (PC)

- 2024-10-30 - [Telemedicine Equipment](#)


We recommend using [ThinkLabs One Stethoscope](#) with VSee.

Turn ON the stethoscope and open the VSee Messenger app.


Set up the VSee Messenger app.

1. Click on the **Gear icon/Settings**.
1. Click on **Audio**.
2. Add a secondary microphone and select from the dropdown - **Stethoscope**.
3. Under **Device**, select from the dropdown list the ThinkLabs One Stethoscope.

Note: The mic name may vary depending on your device but it will always include the word "USB".

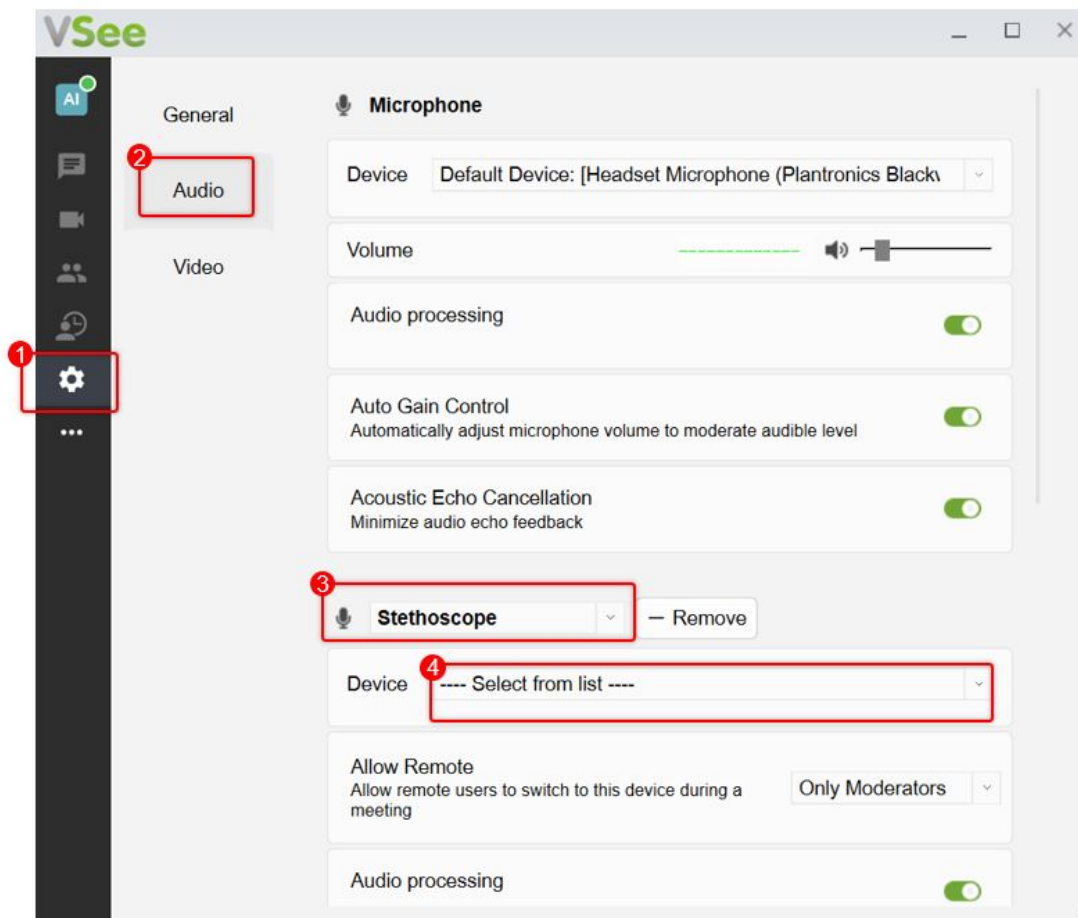

1. On the **Allow Remote** field, select from the dropdown list - **Only Moderators**. This is important so that patients cannot turn ON/OFF the secondary microphone during the call.
1. On the **Audio processing** field, toggle ON.

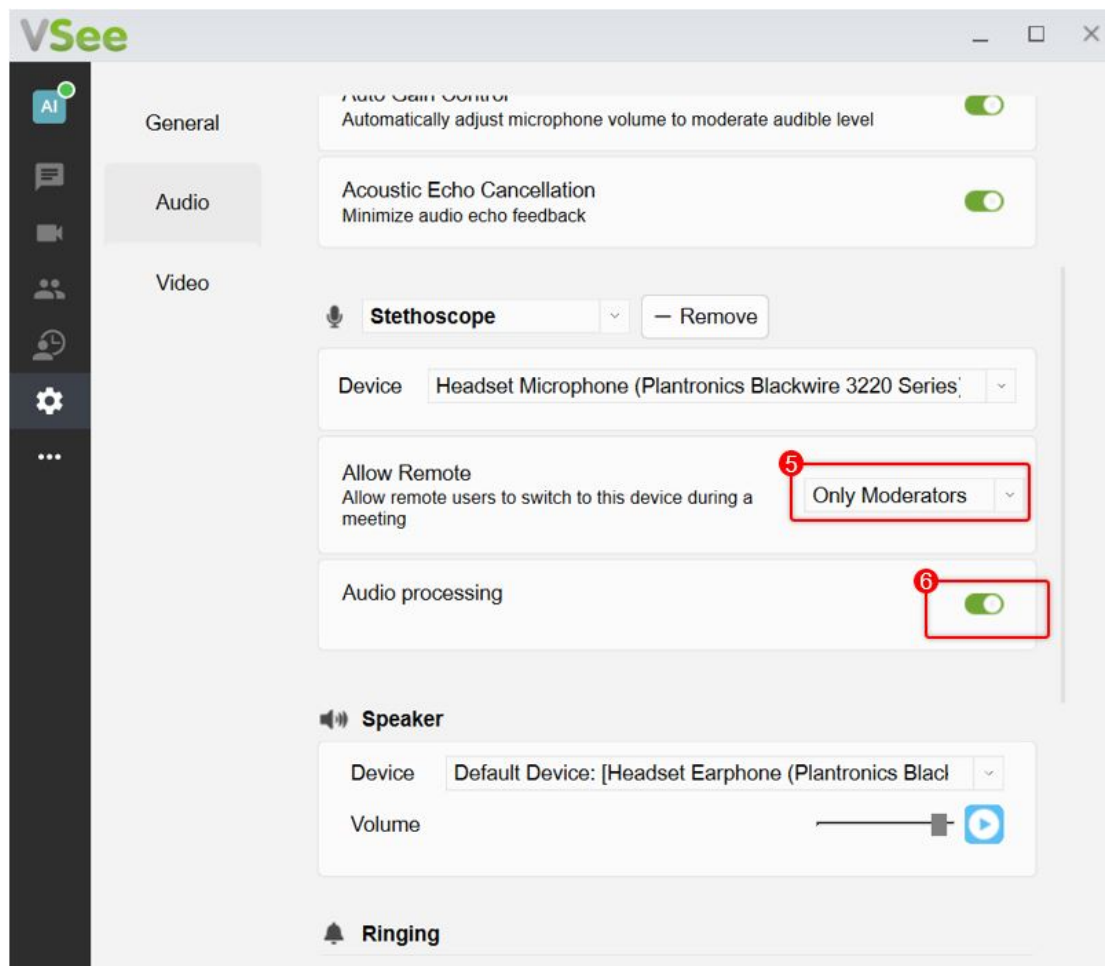

Please refer to the stethoscope manual for details on how to further optimize the stethoscope's sound quality.

Contact us at help@vsee.com for questions or assistance with your stethoscope.