


Stethoscope Mode in VSee Messenger


- 2021-02-15 - VSee Messenger (Providers and Patients)

We recommend using [ThinkLabs One Stethoscope](#) with VSee.


1. Turn on the stethoscope.
2. Open the VSee Messenger app.
3. Click on the **gear icon** > **Audio** > **Microphone** > Select **Microphone (Generic USB Audio Device)** from the dropdown list. The mic name may vary depending on your device but it will always include the word "USB".

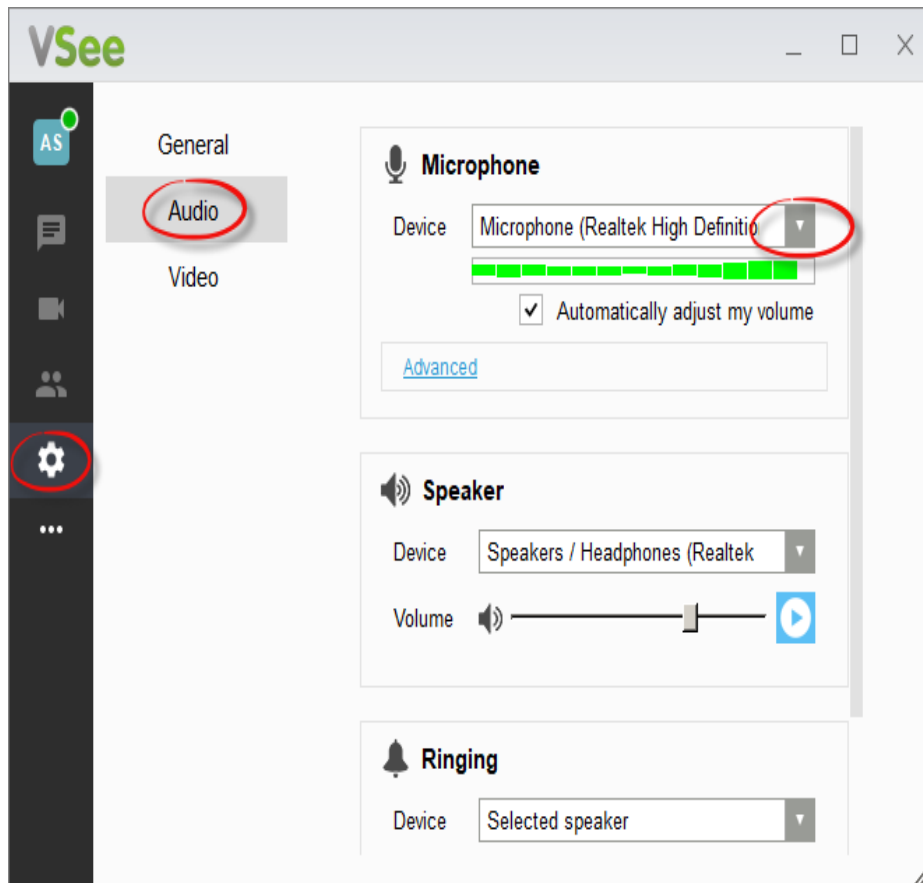

4. Tap on the stethoscope until the microphone's green volume wave moves.
5. Still under **Microphone**, click **Advanced** > Check **Stethoscope**

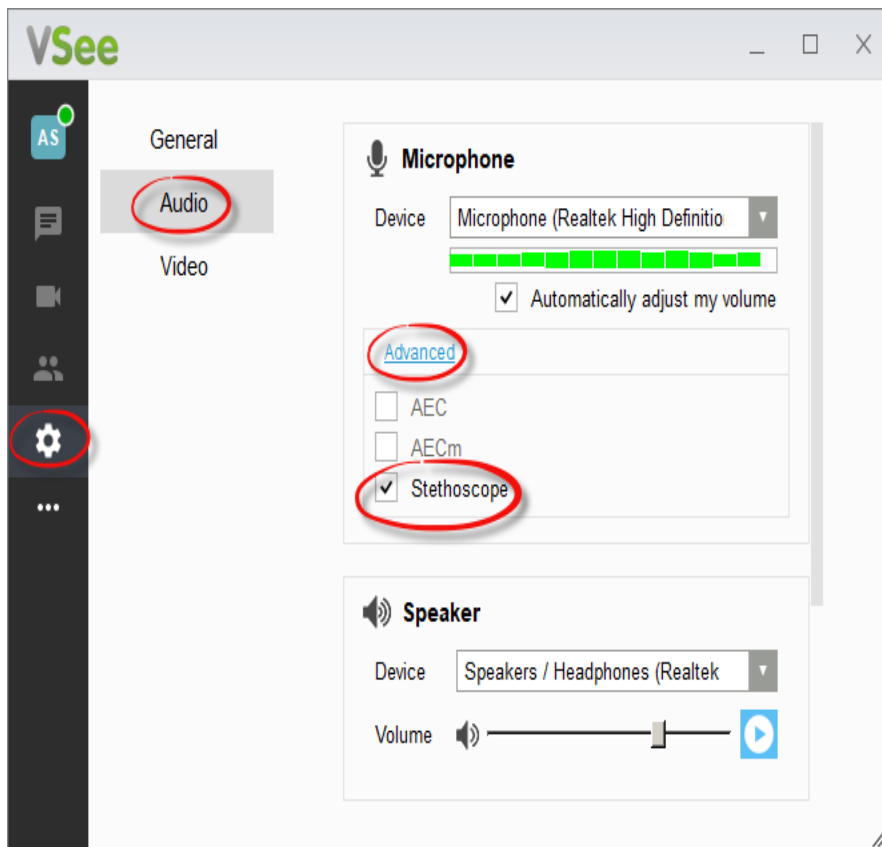

6. A warning message will be shown. Click **OK** to agree.

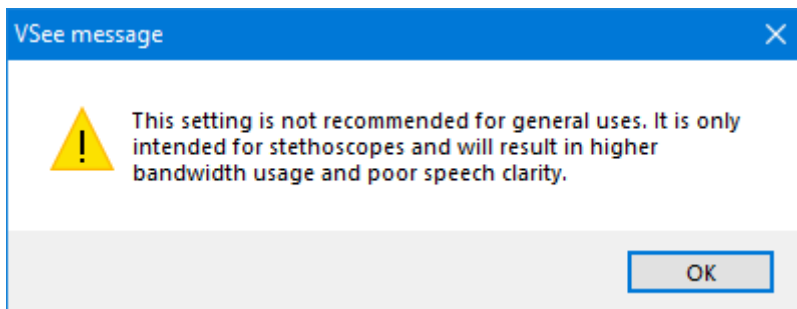

7. Please refer to the stethoscope manual for details on how to further optimize the stethoscope's sound quality.

Please contact us at help@vsee.com for questions or assistance with your stethoscope.