

Knowledgebase > Telemedicine Equipment > Stethoscope Mode in VSee Messenger (PC)

Stethoscope Mode in VSee Messenger (PC) - 2024-10-30 - Telemedicine Equipment

We recommend using ThinkLabs One Stethoscope with VSee.

Turn ON the stethoscope and open the VSee Messenger app.

Set up the VSee Messenger app.

- 1. Click on the **Gear icon/Settings**.
- 1. Click on Audio.
- 2. Add a secondary microphone and select from the dropdown **Stethoscope**.
- 3. Under **Device**, select from the dropdown list the ThinkLabs One Stethoscope.

Note: The mic name may vary depending on your device but it will always include the word "USB".

VSee		-		×
General	Microphone			
Audio	Device Default Device: [Headset Microphone (Plantronics Black	Ÿ		
Video	Volume		•	
	Audio processing	•		
*	Auto Gain Control Automatically adjust microphone volume to moderate audible level			
	Acoustic Echo Cancellation Minimize audio echo feedback			
	3 Stethoscope - Remove			
	Device 4 Select from list	Ŷ	J	
	Allow Remote Allow remote users to switch to this device during a meeting	s v	1	
	Audio processing			

- On the Allow Remote field, select from the dropdown list Only Moderators. This
 is important so that patients cannot turn ON/OFF the secondary microphone during
 the call.
- 1. On the **Audio processing** field, toggle ON.

VSe	e	_ 🗆 X
	General	Automatically adjust microphone volume to moderate audible level
₽	Audio	Acoustic Echo Cancellation Minimize audio echo feedback
**	Video	Stethoscope - Remove
\$		Device Headset Microphone (Plantronics Blackwire 3220 Series)
		Allow Remote Allow remote users to switch to this device during a meeting
		Audio processing
		■(+) Speaker
		Device Default Device: [Headset Earphone (Plantronics Black ~
		Volume
		Ringing

Please refer to the stethoscope manual for details on how to further optimize the stethoscope's sound quality.

Contact us at <u>help@vsee.com</u> for questions or assistance with your stethoscope.