



Knowledgebase > Telemedicine Equipment > Stethoscope Mode in VSee Messenger (iOS)

---

## Stethoscope Mode in VSee Messenger (iOS)

Chi - 2024-10-30 - Telemedicine Equipment

On iOS devices, the stethoscope audio can be switched ON/OFF through the device's settings:

1. Click on the **Gear icon/Settings**.
1. Under Settings, scroll down and select **VSee Messenger**.

1:18

67

## Settings

Instones

Keynote

LetsView

Messenger

Numbers

Pages

Record it!

SCRL

Slack

SNOW

Spotify

TestFlight

Timepage

VSee

VSee Clinic

Welch Allyn Home

2

1. Scroll down. Under Advanced, toggle ON - **External Mic is Stethoscope**.

Note: Switch ON only when a stethoscope is connected to the device.

1:19

67

< Settings

VSee

PREFERRED LANGUAGE



Language

English >

VSEE SETTINGS

GENERAL

Hide Offline Contacts



Show Network Warnings



Auto Accept All Calls



Mute Chat Notifications When  
In Call



ADVANCED

External Mic is Stethoscope



3

This setting will automatically use stethoscope audio mode whenever an external mic is connected. This should only be used with stethoscopes as it will result in higher bandwidth usage and poor speech clarity with a normal microphone.

ABOUT

Version

4.22.0 (51358)

Acknowledgements



#### Tags

stethoscope

stethoscope mode