



Knowledgebase > Telemedicine Equipment > Stethoscope Mode in VSee Messenger (iOS)

Stethoscope Mode in VSee Messenger (iOS)

Chi - 2024-10-30 - Telemedicine Equipment

On iOS devices, the stethoscope audio can be switched ON/OFF through the device's settings:

1. Click on the **Gear icon/Settings**.

1:19

67

Manila
27°
Cloudy
H:30° L:26°


Weather



FaceTime



Calendar

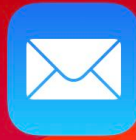

Mail



Notes



Reminders



Clock




TV



Podcasts




App Store

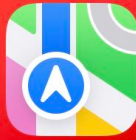

Maps



Health

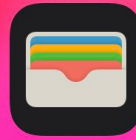

Wallet



Settings



Safari

Search



1. Under Settings, scroll down and select **VSee Messenger**.



1. Scroll down. Under Advanced, toggle ON - **External Mic is Stethoscope**.

Note: Switch ON only when a stethoscope is connected to the device.



Tags

stethoscope

stethoscope mode