



[Knowledgebase](#) > [Telemedicine Equipment](#) > [Stethoscope Mode in VSee Messenger \(iOS\)](#)

Stethoscope Mode in VSee Messenger (iOS)

Chi - 2024-10-30 - [Telemedicine Equipment](#)

On iOS devices, the stethoscope audio can be switched ON/OFF through the device's settings:

1. Click on the **Gear icon/Settings**.

1:19

67

Manila
27°
Cloudy
H:30° L:26°

Weather



FaceTime



Calendar



Mail



Notes



Reminders



Clock




TV



Podcasts



App Store

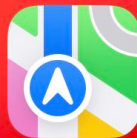

Maps



Health



Wallet



Settings



Safari

Search



1. Under Settings, scroll down and select **VSee Messenger**.



1. Scroll down. Under Advanced, toggle ON - **External Mic is Stethoscope**.

Note: Switch ON only when a stethoscope is connected to the device.



- Tags
- [stethoscope](#)
- [stethoscope mode](#)