



Knowledgebase > Telemedicine Equipment > Stethoscope Mode in VSee Messenger (iOS)

---

## Stethoscope Mode in VSee Messenger (iOS)

Chi - 2024-10-30 - Telemedicine Equipment

On iOS devices, the stethoscope audio can be switched ON/OFF through the device's settings:

1. Click on the **Gear icon/Settings**.
1. Under Settings, scroll down and select **VSee Messenger**.

1:18

67

## Settings

- Instones
- Keynote
- LetsView
- Messenger
- Numbers
- Pages
- Record it!
- SCRL
- Slack
- SNOW
- Spotify
- TestFlight
- Timepage
- V** VSee
- VSee Clinic
- Welch Allyn Home

2

1. Scroll down. Under Advanced, toggle ON - **External Mic is Stethoscope**.

Note: Switch ON only when a stethoscope is connected to the device.

1:19

67

< Settings

VSee

PREFERRED LANGUAGE



Language

English >

VSEE SETTINGS

GENERAL

Hide Offline Contacts



Show Network Warnings



Auto Accept All Calls



Mute Chat Notifications When In Call



ADVANCED

External Mic is Stethoscope



3

This setting will automatically use stethoscope audio mode whenever an external mic is connected. This should only be used with stethoscopes as it will result in higher bandwidth usage and poor speech clarity with a normal microphone.

ABOUT

Version

4.22.0 (51358)

Acknowledgements



Tags

stethoscope

stethoscope mode